
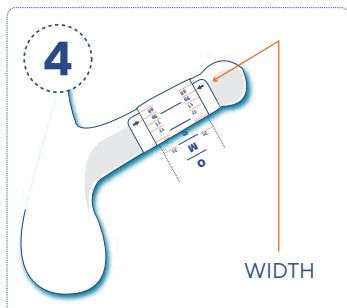
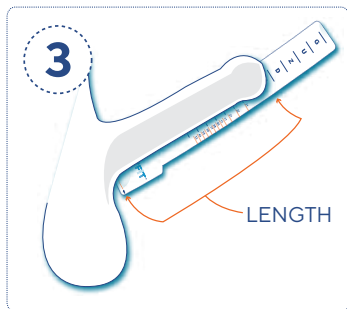
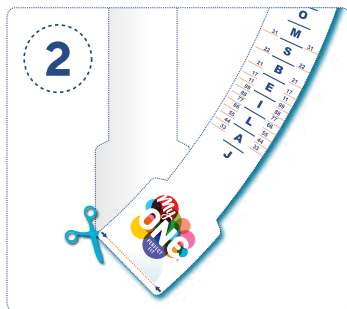


# Introducing myONE®, the world's first perfect-fit condom with 60 different sizes for maximum pleasure & comfort.

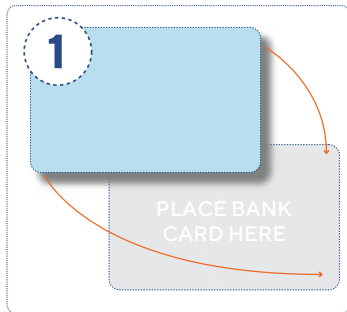

## FitKit™ Instructions

**STEP 1: Print your fit kit.** It's important that the Fit Kit print correctly so you are sized properly. To verify, place a bank card onto the illustration below. If it fits, so will you. Troubleshooting printer issues: Printer scaling is the most common reason the FitKit does not print correctly. Be sure to turn printer scaling off and verify that you're printing the document at 100% size.

**STEP 2: Cut out the measuring tool.** Letters are for length and numbers are for width.

**STEP 3: Find your length.** Place the measuring tool under your erection with the wide end toward your body. Be sure that the edge of the Fit Kit is at the base of your penis, ensuring that the entire shaft will be covered. Your length is the letter that you see above the tip of your penis.

**STEP 4: Find your width.** Wrap the measuring tool gently around the middle of your erection. Your width is shown where the arrows meet the numbers. If your size is in between two numbers, select the number that is closest.

**STEP 5: Determine your size.** Combine your Length Letter with your Width Number to determine your myONE condom size. Visit [myonecondoms.com](http://myonecondoms.com) to purchase or locate a retailer near you.

*Please note: your myONE size is a suggested size based on your measurements. You may want to choose a slightly different size based on your personal preference. If your combination is not shown, to make the best choice choose a nearby size. It's better that a condom has more length or less width rather than be too wide or short. A proper fit is both comfortable and held snugly in position. Don't be afraid to try more than one code combination!*

